




Lancing Device Instructions for Use

Read all instructions carefully before use.

Model: HH-XIII-T

Main Structure of Lancing Device:

- A. Device Cap
- B. Depth Adjustment Dial
- C. Hub (Connection Ring)
- D. Release Button
- E. Device Body
- F. Cocking Handle
- G. Lancet Holder
- H. Lancet Ejector
- J. Lancet Protective Disk

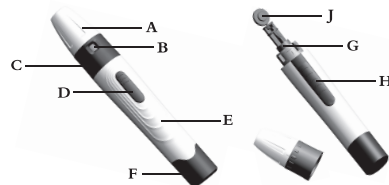

Adjustable Depth Settings

The following depth settings are for reference. Results may vary based on the brand of lancet used. Please adjust the depth according to your skin type and the required blood volume.


Level	Level 1	Level 2	Level 3	Level 4	Level 5
Penetration depth range	0.60mm	1.00mm	1.40mm	1.70mm	2.00mm
Deviation: ± 0.30 mm					

General Information


- **Indications for Use:** This lancing device is intended for use with sterile lancets to collect capillary blood samples from the fingertip for glucose testing.
- **Single-Patient Use Only:** This device is intended for use by a single individual and must never be shared, even with family members.
- **Intended Population:** Suitable for individuals 6 months of age and older.
- **Storage:** Store in a cool, dry, and well-ventilated area. Keep away from corrosive gases and direct sunlight.
- **Service Life:** 5,000 uses or 10 years, whichever comes first.

Operation Instructions


Read all instructions carefully and wash hands before and after each use.

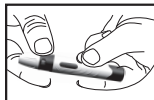


1. Remove the device cap.

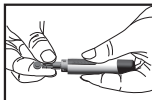


2. Insert a new lancet firmly into the holder until it is fully seated.

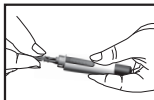


3. Twist off the lancet's protective disk. Save the disk for safe disposal.

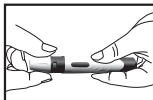


4. Replace the device cap and ensure it is snapped into place.

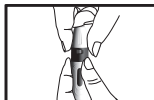


5. Select your desired depth setting. (Mid-level is recommended for first-time use).

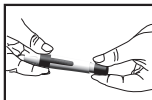


6. Pull back the cocking handle until it clicks.

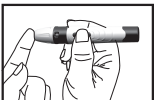


7. Hold the device firmly against the testing site and press the release button.

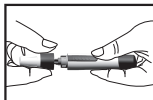

8. Remove the device cap and carefully recap the lancet with the protective disk.

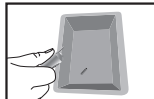

9. Slide the lancet ejector forward to discard the used lancet into a sharps container.

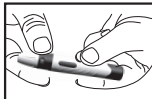

10. Replace the device cap.

Note:

- Adjust the depth level according to your skin type; higher numbers increase penetration depth.
- Hold the device vertically and firmly against the skin to ensure an adequate blood drop.

Warnings:

1. **Safety First:** Do not press the release button while the device cap is removed.
2. **Immediate Disposal:** Always remove and discard the lancet immediately after each use. Do not store the lancet inside the device.
3. **Single-Patient Use Only:** This device is intended for a single user only. Do not share the device or lancets with anyone, including family members.
4. **Care Instructions:** Do not immerse the device in any liquid.
5. **Hygiene:** Clean and disinfect the device regularly according to the "Cleaning and Disinfection" instructions.
6. **Discard After Use:** Used lancets must be safely disposed of in a sharps container immediately after a single use.

7. **Infection Risk:** Sharing the device poses a high risk of transmitting bloodborne pathogens (e.g., HIV, Hepatitis). Cleaning and disinfection reduce the risk of local infection but do not make the device safe for use by more than one person.
8. **Adult Supervision:** For users under 18, the device should be used by an adult or under strict adult supervision.
9. **Pediatric Safety:** To prevent injury, always start at the lowest depth setting when testing children. Consult a healthcare professional before using the device on a child for the first time.

Cleaning and Disinfection:

To prevent the spread of pathogens and ensure the device works properly, please follow these instructions carefully. Failure to do so may damage the device.

- **Target Areas:** The entire surface of the lancing device and the device cap.
- **Important:** Do not allow moisture to enter any openings. Use a Super Sani-Cloth for both cleaning and disinfection.

Steps for Cleaning and Disinfection:

1. **Wash hands** thoroughly with soap and water; dry completely.
2. **Cleaning:** Use a fresh Super Sani-Cloth to wipe the entire surface of the lancing device and the inside of the cap to remove all visible dirt or debris.
3. **Disinfection:** Use a **new** cloth to wipe the device and cap again. Ensure the treated surfaces remain **visibly wet for 2 full minutes**. Let the device air dry.
4. **Wash hands** again after handling the device.

Notes:

- **Frequency:** The device should be cleaned and disinfected before and after each use. If blood contacts the device, disinfect it immediately.
- **Heavily Soiled Devices:** If visible soil remains after cleaning, repeat steps 2 and 3.
- **Skin Preparation:** If the puncture site is visibly dirty, wash with soap and water and dry thoroughly. Always apply alcohol to the skin and let it air dry before sampling.

Caution:

- **General Safety:** Read all instructions carefully before use. Avoid accidental injury by handling the device and lancets with care. Do not point the lancing device at yourself or others.
- **Sterility & Expiration:** Only use lancets within their expiration date. Using expired lancets may lead to infection as sterility is no longer guaranteed.
- **Third-Party Assistance:** If someone else is assisting you with testing, they must wear protective gloves. The device must be cleaned and disinfected before being handled by the assistant.
- **Infection Control:** The lancing device and lancets are for single-patient use only. Never share them, even with family members or healthcare professionals testing multiple people. If the device comes into contact with blood, it poses a risk of cross-infection.
- **Device Integrity:**
 - If the device or lancet is dropped, crushed, or damaged (e.g., visible cracks), do not use it. Discard and replace it immediately.
 - If the device is dropped with a lancet inside, the lancet may become loose or protrude, posing a needle-stick hazard. Handle with extreme caution.
- **Choking Hazard:** Keep the device and all small components out of reach of children under 3 years of age.
- **Compatibility:** Only use compatible lancets with this device. If using third-party sterile lancets, verify compatibility before use.
- **Professional Advice:** Consult a healthcare professional before use for proper guidance.

Troubleshooting

Issue	Possible Cause	Solution
Blood sample is insufficient.	Depth setting is too shallow.	Increase the depth level on the adjustment dial.
The device does not fire.	The cocking handle is not pulled back.	Pull the cocking handle back until it clicks.
	The release button was pressed too lightly.	Press the release button firmly.
The lancet is hard to insert.	The lancet is not compatible or is bent.	Check compatibility or use a new, straight lancet.
Painful testing.	Depth setting is too deep.	Decrease the depth level.
	The lancet is blunt (reused).	Always use a new, sterile lancet for each test.

Specifications

• Product Name	Lancing Device
• Adjustment Range	5 Depth Settings (0.6mm – 2.00mm)
• Operating Temperature	50°F to 104°F (10°C to 40°C)
• Storage Temperature	-4°F to 131°F (-20°C to 55°C)
• Humidity Range	10% – 85% RH (Non-condensing)
• Service Life	5,000 uses or 10 years, whichever comes first.
• Compatibility	Compatible with most standard universal lancets.

Symbol Descriptions

	Manufacturer		Consult instructions for use		Date of manufacture
	Caution		Batch code		

Manufactured for:

GL DIABETES CORPORATION
9676 Oviedo St., San Diego,
CA 92129-3828, USA
Customer Service: 1-877-288-2722
Email: sales@gl-diabetes.com
Website: www.gl-diabetes.com